



# Recommended Edwards SAPIEN 3 Transcatheter Pulmonary Valve System with Alterra Adaptive Present

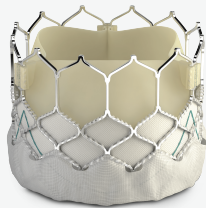

Edwards SAPIEN 3 transcatheter heart valve

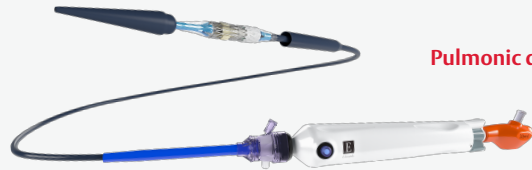

Pulmonic delivery system




Inflation device



Edwards crimp stopper



Edwards crimper

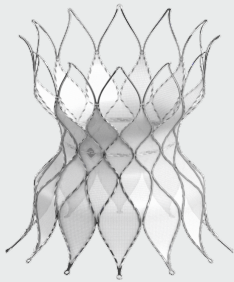

Alterra adaptive present



Alterra delivery system



Edwards eSheath introducer set

## Additional components needed

- Two THV rinsing bowls  $\geq$  500 mL sterile physiologic saline
- Two qualcrimp crimping accessory rinsing bowls  $\geq$  100 mL sterile physiologic saline
- Flushing catheter bowl  $\geq$  500 mL sterile heparinized saline
- Contrast solution bowl of 15% contrast solution (85 mL saline + 15 mL contrast)
- 20 cc or larger luer lock syringe for flushing catheters
- 50 cc or larger luer lock syringe for delivery system preparation
- Sterile forceps
- Sterile scissors
- High pressure 3-way rotating stopcock

# Recommended site equipment

## Balloons

Manufacturer	Product Name	Diameter Range	Length Ranges
B. Braun	PTS - X	25 – 40 mm diameter	3, 4, & 5 cm
B. Braun/NuMed	Z-Med/Z Med II	24 – 30 mm diameter	2 – 6 cm
Medtronic	Reliant	10-46 mm diameter	NA
Cook Medical	Coda	32-46 mm diameter	NA

## Wires

Manufacturer	Product Name	Size	Range
Cook Medical	Lunderquist	0.035	260 - 300 cm
Medtronic	Confida Wire	0.035	260 cm
Terumo	Glide wires	0.035	180 cm
Boston Scientific	Amplatz Super Stiff	0.035	260 cm
Abbott	Versacore	0.035	260 - 300 cm

## Sheaths

Manufacturer	Product Name	French	Length
Gore	DrySeal	20 - 26 Fr	33, 45, 65* cm

## Catheters

Manufacturer	Product Name	Size
Arrow	Balloon wedge pressure catheter	7, 8 Fr
Various	Marker pigtail	5 Fr
B. Braun	Multi-Track	8 Fr
Arrow	Berman angiographic catheter	7 Fr

## Ultrasound

Manufacturer	Product Name	Size
Siemens	AcuNav	8 or 10 Fr
St Jude	ViewFlex	9 Fr

\*65 cm Gore should not be used with the Alterra delivery system.

**CAUTION: Federal (United States) law restricts these devices to sale by or on the order of a physician. See Instructions for Use for full prescribing information, including indications, contraindications, warnings, precautions, and adverse events.**

Edwards, Edwards Lifesciences, the stylized E logo, Alterra, Carpentier-Edwards, EDWARDS COMMANDER, Edwards SAPIEN, Edwards SAPIEN 3, eSheath, NovaFlex, NovaFlex+, Qualcrimp, SAPIEN, SAPIEN 3 and TheraFix are trademarks or service marks of Edwards Lifesciences Corporation. All other trademarks or service marks are the property of their respective owners.

© 2022 Edwards Lifesciences Corporation. All rights reserved. DOC-0187998 Rev A

Edwards Lifesciences • One Edwards Way, Irvine CA 92614 USA • edwards.com



**Edwards Proprietary. Do Not Reproduce or Distribute.**