



See the Difference

Get a quick look at the treatment options to help your patient better understand them.

 SAVR	or	TAVI 
Available for all surgical risk patients (except prohibitive risk) ¹		Available for severe, symptomatic calcific aortic stenosis patients, independent of surgical risk
More invasive procedure ²		Less invasive procedure ²
Requires sternotomy ²		The most common approach is the transfemoral approach. ²
Requires stopping the heart and connecting the patient to a blood-pumping machine ²		Does not require stopping the heart ²
Requires general anesthesia		May be performed using conscious sedation
On average, may last ~3 hours*		On average, may last ~1 hour*
Average hospital stay may be 7 days*		Average hospital stay may be 3 days*

Guide your patients to the best chance at a superior outcome*. Partner with a local Heart Team.

*PARTNER 3 Trial proved SAPIEN 3 TAVI is superior to surgery on the primary endpoint (all-cause death, all stroke, and rehospitalization) and multiple pre-specified secondary endpoints for low-risk patients.

References: 1. Nishimura R, Otto C, Bonow R, et al. 2017 AHA/ACC focused update of the 2014 AHA/ACC guideline for the management of patients with valvular heart disease: executive summary. *J Am Coll Cardiol.* 2017;135(25):e1159-e1195. 2. SAPIEN 3 Patient Brochure for Severe Aortic Stenosis Patients; DOC-0124393.

Edwards SAPIEN 3 THV System and Edwards SAPIEN 3 Ultra THV System

Indications: The Edwards SAPIEN 3 Transcatheter Heart Valve System and Edwards SAPIEN 3 Ultra Transcatheter Heart Valve System are indicated for relief of aortic stenosis in patients with symptomatic heart disease due to severe native calcific aortic stenosis who are judged by a Heart Team, including a cardiac surgeon, to be appropriate for the transcatheter heart valve replacement therapy.

Contraindications: The valves and delivery systems are contraindicated in patients who cannot tolerate an anticoagulation/antiplatelet regimen or who have active bacterial endocarditis or other active infections.

Warnings: Observation of the pacing lead throughout the procedure is essential to avoid the potential risk of pacing lead perforation. There may be an increased risk of stroke in transcatheter aortic valve replacement procedures, as compared to balloon aortic valvuloplasty or other standard treatments in high or greater risk patients. Incorrect sizing of the valve may lead to paravalvular leak, migration, embolization, residual gradient (patient-prosthesis mismatch), and/or annular rupture. Accelerated deterioration of the valve due to calcific degeneration may occur in children, adolescents, or young adults and in patients with an altered calcium metabolism. Prior to delivery, the valve must remain hydrated at all times and cannot be exposed to solutions other than its shipping storage solution and sterile physiologic rinsing solution. Valve leaflets mishandled or damaged during any part of the procedure will require replacement of the valve. Caution should be exercised in implanting a valve in patients with clinically significant coronary artery disease. Patients with pre-existing bioprostheses should be carefully assessed prior to implantation of the valve to ensure proper valve positioning and deployment. Do not use the valve if the tamper-evident seal is broken, the storage solution does not completely cover the valve, the temperature indicator has been activated, the valve is damaged, or the expiration date has elapsed. Do not mishandle the delivery system or use it if the packaging or any components are not sterile, have been opened or are damaged (e.g., kinked or stretched), or if the expiration date has elapsed. Use of excessive contrast media may lead to renal failure. Measure the patient's creatinine level prior to the procedure. Contrast media usage should be monitored. Patient injury could occur if the delivery system is not un-flexed prior to removal. Care should be exercised in patients with hypersensitivities to cobalt, nickel, chromium, molybdenum, titanium, manganese, silicon, and/or polymeric materials. The procedure should be conducted under fluoroscopic guidance. Some fluoroscopically guided procedures are associated with a risk of radiation injury to the skin. These injuries may be painful, disfiguring, and long-lasting. Valve recipients should be maintained on anticoagulant/antiplatelet therapy, except when contraindicated, as determined by their physician. This device has not been tested for use without anticoagulation. Do not add or apply antibiotics to the storage solution, rinse solution, or to the valve. Balloon valvuloplasty should be avoided in the treatment of failing bioprostheses as this may result in embolization of bioprosthesis material and mechanical disruption of the valve leaflets. Failure to use slow, controlled inflation and prescribed nominal inflation volumes may result in balloon rupture, and lead to patient death or serious injuries associated with difficulty retrieving the delivery system and surgical intervention.

Precautions: Safety, effectiveness, and durability have not been established for THV-in-THV procedures. Long-term durability has not been established for the valve. Regular medical follow-up is advised to evaluate valve performance. Glutaraldehyde may cause irritation of the skin, eyes, nose, and throat. Avoid prolonged or repeated exposure to, or breathing of, the solution. Use only with adequate ventilation. If skin contact occurs, immediately flush the affected area with water; in the event of contact with eyes, seek immediate medical attention. For more information about glutaraldehyde exposure, refer to the Safety Data Sheet available from Edwards Lifesciences. To maintain proper valve leaflet coaptation, do not overinflate the deployment balloon. Appropriate antibiotic prophylaxis is recommended post-procedure in patients at risk for prosthetic valve infection and endocarditis. Additional precautions for transseptal replacement of a failed mitral valve bioprosthesis include, the presence of devices or thrombus or other abnormalities in the caval vein precluding safe transvenous femoral access for transseptal approach; and the presence of an Atrial Septal Occluder Device or calcium or abnormalities in the atrial septum preventing safe transseptal access. Special care must be exercised in mitral valve replacement if chordal preservation techniques were used in the primary implantation to avoid entrapment of the subvalvular apparatus. Safety and effectiveness have not been established for patients with the following characteristics/comorbidities: non-calcified aortic annulus; severe ventricular dysfunction with ejection fraction < 20%; congenital unicuspid aortic valve; congenital bicuspid aortic valve in low surgical risk patients; mixed aortic valve disease (aortic stenosis and aortic regurgitation with predominant aortic regurgitation > 3+); pre-existing prosthetic ring in any position; severe mitral annular calcification (MAC); severe (> 3+) mitral insufficiency, or Gorlin syndrome; blood dyscrasias defined as leukopenia (WBC < 3000 cells/mL), acute anemia (Hb < 9 g/dL), thrombocytopenia (platelet count < 50,000 cells/mL), or history of bleeding diathesis or coagulopathy; hypertrophic cardiomyopathy with or without obstruction (HOCM); echocardiographic evidence of intracardiac mass, thrombus, or vegetation; a known hypersensitivity or contraindication to aspirin, heparin, ticlopidine (Ticlid), or clopidogrel (Plavix), or sensitivity to contrast media, which cannot be adequately premedicated; significant aortic disease, including abdominal aortic or thoracic aneurysm defined as maximal luminal diameter 5 cm or greater, marked tortuosity (hyperacute bend), aortic arch atheroma (especially if thick [> 5 mm], protruding, or ulcerated) or narrowing (especially with calcification and surface irregularities) of the abdominal or thoracic aorta, severe "unfolding" and tortuosity of the thoracic aorta; access characteristics that would preclude safe placement of the 14F or 16F Edwards eSheath introducer set or the 14F Axela sheath, such as severe obstructive calcification, severe tortuosity, or diameter less than 5.5 mm (14F Axela or 14F eSheath introducer set) or 6 mm (16F eSheath introducer set or 14F Axela in subclavian access); excessive calcification at access site; bulky calcified aortic valve leaflets in close proximity to coronary ostia; a concomitant paravalvular leak where the failing bioprosthesis is not securely fixed in the native annulus or is not structurally intact (e.g., wireframe fracture); or a partially detached leaflet of the failing bioprosthesis that in the aortic position may obstruct a coronary ostium. Residual mean gradient may be higher in a "THV-in-failing bioprosthesis" configuration than that observed following implantation of the valve inside a native aortic annulus using the same size device. Patients with elevated mean gradient post procedure should be carefully followed. It is important that the manufacturer, model and size of the preexisting bioprosthetic valve be determined, so that the appropriate valve can be implanted and a prosthesis-patient mismatch be avoided. Additionally, pre-procedure imaging modalities must be employed to make as accurate a determination of the inner diameter as possible.

pseudoaneurysm; reoperation; ischemia or nerve injury; restenosis; pulmonary edema; pleural effusion; bleeding; anemia; abnormal lab values (including electrolyte imbalance); hypertension or hypotension; allergic reaction to anesthesia, contrast media, or device materials; hematoma; syncope; pain or changes at the access site; exercise intolerance or weakness; inflammation; angina; heart murmur; and fever. Additional potential risks associated with the use of the valve, delivery system, and/or accessories include: cardiac arrest; cardiogenic shock; emergency cardiac surgery; cardiac failure or low cardiac output; coronary flow obstruction/transvalvular flow disturbance; device thrombosis requiring intervention; valve thrombosis; device embolization; device migration or malposition requiring intervention; left ventricular outflow tract obstruction; valve deployment in unintended location; valve stenosis; structural valve deterioration (wear, fracture, calcification, leaflet tear/tearing from the stent posts, leaflet retraction, suture line disruption of components of a prosthetic valve, thickening, stenosis); device degeneration; paravalvular or transvalvular leak; valve regurgitation; hemolysis; injury to the mitral valve; device explants; mediastinitis; mediastinal bleeding; nonstructural dysfunction; mechanical failure of delivery system and/or accessories; and non-emergent reoperation.

Edwards Axela Sheath

Indications: The Edwards Axela sheath is indicated for the introduction and removal of devices used with the Edwards SAPIEN 3 Ultra delivery system.

Contraindications: There are no known contraindications.

Warnings: The devices are designed, intended, and distributed for single use only. Do not resterilize or reuse the devices. There are no data to support the sterility, nonpyrogenicity, and functionality of the devices after reprocessing.

Potential Adverse Events: Complications associated with standard catheterization and use of angiography include, but are not limited to, injury including perforation or dissection of vessels, thrombosis and/or plaque dislodgement which may result in emboli formation, distal vessel obstruction, hemorrhage, infection, and/or death.

Edwards eSheath

Indications: The Edwards eSheath introducer set is indicated for the introduction and removal of devices used with the Edwards SAPIEN 3 and the Edwards SAPIEN 3 Ultra transcatheter heart valves.

Contraindications: There are no known contraindications.

Warnings: The devices are designed, intended, and distributed for single use only. Do not resterilize or reuse the devices. There is no data to support the sterility, nonpyrogenicity, and functionality of the devices after reprocessing. The Edwards eSheath introducer set must be used with a compatible 0.035" (0.89 mm) guidewire to prevent vessel injury.

Precautions: Caution should be used in vessels that have diameters less than 5.5 mm or 6 mm as it may preclude safe placement of the 14F and 16F Edwards eSheath introducer set respectively. Use caution in tortuous or calcified vessels that would prevent safe entry of the introducer set. Do not use the Edwards eSheath introducer set if the packaging sterile barriers and any components have been opened or damaged. When inserting, manipulating or withdrawing a device through the sheath, always maintain sheath position. When puncturing, suturing or incising the tissue near the sheath, use caution to avoid damage to the sheath.

Potential Adverse Events: Complications associated with standard catheterization and use of angiography include, but are not limited to, injury including perforation or dissection of vessels, thrombosis and/or plaque dislodgement which may result in emboli formation, distal vessel obstruction, hemorrhage, infection, and/or death.

Edwards Crimper

Indications: The Edwards crimper is indicated for use in preparing the Edwards SAPIEN 3 Ultra transcatheter heart valve and the Edwards SAPIEN 3 transcatheter heart valve for implantation.

Warnings: The devices are designed, intended, and distributed for single use only. Do not resterilize or reuse the devices. There are no data to support the sterility, nonpyrogenicity, and functionality of the devices after reprocessing.

Precautions: For special considerations associated with the use of the Edwards crimper prior to THV implantation, refer to the THV Instructions for Use.

Potential Adverse Events: There are no known potential adverse events associated with the Edwards crimper.



The Medical Technology Association of Canada ("Medtech Canada") is dedicated to advancing healthcare through innovative technologies, devices and diagnostics ("technologies"). Medtech Canada believes that access to high quality, cost-effective healthcare technology is paramount to the improvement of patient care. Medtech Canada represents companies that design, develop, manufacture and market medical technologies and related services used in the treatment, mitigation, diagnosis or prevention of a disease or abnormal physical condition.

The Medtech Canada Code of Conduct is designed to ensure that all relationships between Companies and Healthcare Professionals are governed by the highest ethical standards. By adhering to these carefully developed guidelines, Companies create a win-win situation. Their vital relationships with Healthcare Professionals are safeguarded, legitimate business practices are affirmed and patients across Canada benefit from vital and innovative collaboration between Healthcare Professionals and the medical technology industry.

For a copy of the Medtech Canada Code of Conduct visit: www.medtechcanada.org/page/code

Edwards, Edwards Lifesciences, the stylized E logo, Edwards Commander, Edwards SAPIEN, Edwards SAPIEN XT, Edwards SAPIEN 3, Edwards SAPIEN 3 Ultra, PARTNER, PARTNER 3, SAPIEN, SAPIEN XT, SAPIEN 3, and SAPIEN 3 Ultra are trademarks or service marks of Edwards Lifesciences Corporation. All other trademarks are the property of their respective owners.

