



# The strategy for lifetime management

Edwards SAPIEN 3 platform



Edwards

# What's your strategy for lifetime management?

With patients living longer and having high expectations for their quality of life, lifetime management is increasingly important.

**The first valve choice matters.** From outstanding outcomes and excellent durability\* today to facilitating future interventions tomorrow, the SAPIEN platform is designed to deliver a TAVR experience an implanter can count on.

Life

Consistently demonstrating the results you need for the outcomes that matter

Time

A comprehensive approach to durability that addresses the key drivers of reintervention

Management

Making future options possible



\*Propensity-matched analysis of intermediate-risk patients using VARC-3 definitions of structural valve deterioration (SVD) and SVD-related bioprosthetic valve failure (BVF) at 5 years.

# Life

## Consistently demonstrating the results you need for the outcomes that matter

SAPIEN 3 transcatheter aortic valve replacement (TAVR) is proven superior\* to surgery in low-risk patients at 1 year and proven equally effective at 5 years<sup>1,2</sup>

### PARTNER 3 low-risk trial clinical events at 30 days, 1 year, and 5 years

	30 days		1 year			5 years		
	TAVR (n = 496)	Surgery (n = 454)	TAVR (n = 496)	Surgery (n = 454)	P-value	TAVR (n = 496)	Surgery (n = 454)	P-value
All-cause death	0.4%	1.1%	1.0%	2.5%	<0.09	10.0%	8.2%	0.35
All stroke	0.6%	2.4%	1.2%	3.1%	0.04	5.8%	6.4%	0.60
Rehospitalization*	3.4%	6.5%	7.3%	11.0%	0.046	13.7%	17.4%	0.09

### Consistently strong procedural outcomes for a variety of patient morphologies

Excellent clinical outcomes in low-risk bicuspid patients.<sup>3</sup>

Consistently strong outcomes across indicated annular sizes.<sup>4</sup>

\* The PARTNER 3 Trial, SAPIEN 3 transcatheter aortic valve replacement (TAVR) proven superior to surgery on the primary endpoint of all-cause death, all stroke, and rehospitalization (valve-related or procedure-related and including heart failure) at one year, and multiple pre-specified secondary endpoints in low-risk patients.

PARTNER 3 Low-Risk Trial 5-Year Results - Low rates of cardiovascular mortality through five years (5.5% SAPIEN 3 system to 5.1% SAVR). Low rates of all-cause mortality through five years (10.0% SAPIEN 3 system vs. 8.2% with SAVR). Low rates of disabling stroke through five years (2.9% SAPIEN 3 system to 2.7% SAVR). Low rates of stroke through five years (5.8% SAPIEN 3 system vs. 6.4% SAVR). Lower rates of rehospitalization with the SAPIEN 3 system through five years (13.7% vs. 17.4%) in low-risk patients.

# Time

## SAPIEN valve technology is designed specifically to support valve durability

A unique design built for differentiated outcomes

### Structural valve deterioration


Bovine pericardium collagen structure that **reduces the potential for tissue tear**

Circularity for **high radial strength** and **optimal hemodynamics**

Three independently matched leaflets for **thickness** and **elasticity**

### Nonstructural valve dysfunction

40% taller\*; textured outer skirt for low, real-world rates of paravalvular leak in a propensity-matched analysis<sup>5</sup>

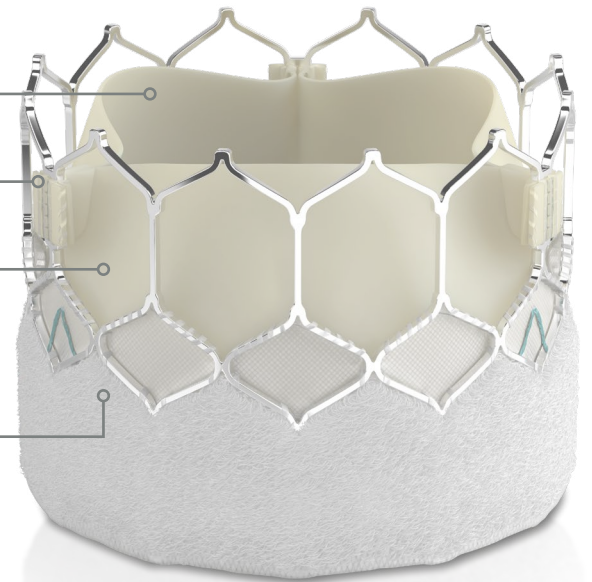

## SAPIEN 3 transcatheter aortic valve replacement has demonstrated excellent durability

**5**  
years

The PARTNER II S3i intermediate-risk trial and PARTNER 3 low-risk trial demonstrated:

- SAPIEN 3 valve **durability similar to surgery** at 5 years<sup>2,6</sup>
- **Low rates of structural valve deterioration (SVD) and SVD-related bioprosthetic valve failure**, similar to surgery at 5 years<sup>2,6</sup>

Real-world data have demonstrated:

- Excellent 5-year SAPIEN valve **durability performance for small-annuli patients**<sup>7</sup>

\*SAPIEN 3 Ultra valve, as compared to SAPIEN 3 valve.

# Management

## Facilitates coronary access, reducing obstacles to future therapy options<sup>5,8</sup>


SAPIEN 3 transcatheter valve system technology is designed to minimize obstacles to future therapy

- Excellent real-world results for continued coronary access

**100%** (n=68/68) successful coronary access post-TAVR<sup>8</sup>

- The **only transcatheter heart valve** indicated for both TAV-in-TAV and TAV-in-SAV procedures\*
  - A valve with excellent safety and procedural outcomes for surgical valve-in-valve procedures\*<sup>9</sup>
  - Outcomes for TAV-in-TAV with the SAPIEN 3 valve are similar to native transcatheter aortic valve replacement<sup>10</sup>

Designed to host future valve interventions

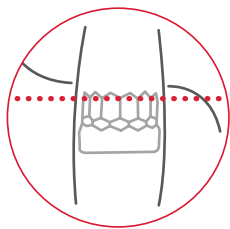

**Low risk plane**

**2.0%** risk of future sinus sequestration<sup>†11</sup>


\*For patients assessed at high risk for surgical replacement.

†Risk of sinus sequestration if (1) prior TAV commissure level was above sinotubular junction (STJ), and (2) the distance between TAV and STJ was <2.0 mm in each coronary sinus.



Edwards SAPIEN 3 platform

**Built for now  
and what's next**

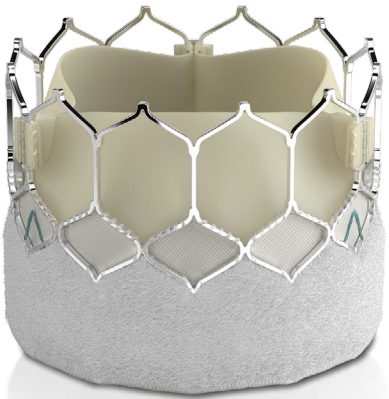

See how the SAPIEN 3 platform works **as your strategy for success in comprehensive lifetime management** at [heartvalves.com](http://heartvalves.com)



Edwards

# Important Safety Information

## **Edwards SAPIEN 3, Edwards SAPIEN 3 Ultra, and Edwards SAPIEN 3 Ultra RESILIA Transcatheter Heart Valve System**

**Indications:** The Edwards SAPIEN 3, SAPIEN 3 Ultra, and SAPIEN 3 Ultra RESILIA Transcatheter Heart Valve system is indicated for relief of aortic stenosis in patients with symptomatic heart disease due to severe native calcific aortic stenosis who are judged by a Heart Team, including a cardiac surgeon, to be appropriate for the transcatheter heart valve replacement therapy.

The Edwards SAPIEN 3, SAPIEN 3 Ultra, and SAPIEN 3 Ultra RESILIA Transcatheter Heart Valve system is indicated for patients with symptomatic heart disease due to failing (stenosed, insufficient, or combined) of a surgical or transcatheter bioprosthetic aortic valve, a surgical bioprosthetic mitral valve, or a native mitral valve with an annuloplasty ring who are judged by a Heart Team, including a cardiac surgeon, to be at high or greater risk for open surgical therapy (i.e., predicted risk of surgical mortality  $\geq 8\%$  at 30 days, based on the Society of Thoracic Surgeons (STS) risk score and other clinical co-morbidities unmeasured by the STS risk calculator).

**Contraindications:** The valves and delivery systems are contraindicated in patients who cannot tolerate an anticoagulation/antiplatelet regimen or who have active bacterial endocarditis or other active infections, or who have significant annuloplasty ring dehiscence.

**Warnings:** Observation of the pacing lead throughout the procedure is essential to avoid the potential risk of pacing lead perforation. There may be an increased risk of stroke in transcatheter aortic valve replacement procedures, as compared to balloon aortic valvuloplasty or other standard treatments in high or greater risk patients. The devices are designed, intended, and distributed for single use only. **Do not resterilize or reuse the devices.** There are no data to support the sterility, nonpyrogenicity, and functionality of the devices after reprocessing. Incorrect sizing of the valve may lead to paravalvular leak, migration, embolization, residual gradient (patient-prosthesis mismatch), and/or annular rupture. Accelerated deterioration of the valve due to calcific degeneration may occur in children, adolescents, or young adults and in patients with an altered calcium metabolism. Prior to delivery, the valve must remain hydrated at all times and cannot be exposed to solutions other than its shipping storage solution and sterile physiologic rinsing solution. Valve leaflets mishandled or damaged during any part of the procedure will require replacement of the valve. Caution should be exercised in implanting a valve in patients with clinically significant coronary artery disease. Patients with pre-existing prostheses should be carefully assessed prior to implantation of the valve to ensure proper valve positioning and deployment. Do not use the valve if the tamper-evident seal is broken or the storage solution does not completely cover the valve (SAPIEN 3 and SAPIEN 3 Ultra only), the temperature indicator has been activated, the valve is damaged, or the expiration date has elapsed. Do not mishandle the delivery system or use it if the packaging or any components are not sterile, have been opened or are damaged (e.g., kinked or stretched), or if the expiration date has elapsed. Use of excessive contrast media may lead to renal failure. Measure the patient's creatinine level prior to the procedure. Contrast media usage should be monitored. Patient injury could occur if the delivery system is not un-flexed prior to removal. Care should be exercised in patients with hypersensitivities to cobalt, nickel, chromium, molybdenum, titanium, manganese, silicon, and/or polymeric materials. The procedure should be conducted under fluoroscopic guidance. Some fluoroscopically guided procedures are associated with a risk of radiation injury to the skin. These injuries may be painful, disfiguring, and long-lasting. Valve recipients should be maintained on anticoagulant/antiplatelet therapy, except when contraindicated, as determined by their physician. This device has not been tested for use without anticoagulation. Do not add or apply antibiotics to the storage solution (SAPIEN 3 and SAPIEN 3 Ultra only), rinse solution, or to the valve. Balloon valvuloplasty should be avoided in the treatment of failing bioprostheses as this may result in embolization of bioprosthesis material and mechanical disruption of the valve leaflets. Do not perform stand-alone balloon aortic valvuloplasty procedures in the INSPIRIS RESILIA aortic valve for the sizes 19-25 mm. This may expand the valve causing aortic incompetence, coronary embolism or annular rupture. Transcatheter valve replacement in mitral annuloplasty rings is not recommended in cases of partial annuloplasty ring dehiscence due to high risk of PVL. Transcatheter valve replacement in mitral annuloplasty rings is not recommended in cases of partial (incomplete) annuloplasty rings in the absence of annular calcium due to increased risk of valve embolization. Transcatheter valve replacement in mitral annuloplasty rings is not recommended in cases of rigid annuloplasty rings due to increased risk of PVL or THV deformation.

**Precautions:** Long-term durability has not been established for the valve. Regular medical follow-up is advised to evaluate valve performance. Limited clinical data are available for transcatheter aortic valve replacement in patients with a congenital bicuspid aortic valve who are deemed to be at low surgical risk. Anatomical characteristics should be considered when using the valve in this population. In addition, patient age should be considered as long-term durability of the valve has not been established. Glutaraldehyde may cause irritation of the skin, eyes, nose, and throat. Avoid prolonged or repeated exposure to, or breathing of, the solution. Use only with adequate ventilation. If skin contact occurs, immediately flush the affected area with water; in the event of contact with eyes, seek immediate medical attention. For more information about glutaraldehyde exposure, refer to the Safety Data Sheet available from Edwards Lifesciences. If a significant increase in resistance occurs when advancing the catheter through the vasculature, stop advancement and investigate the cause of resistance before proceeding. Do not force passage, as this could increase the risk of vascular complications. As compared to SAPIEN 3, system advancement force may be higher with the use of SAPIEN 3 Ultra/SAPIEN 3 Ultra RESILIA THV in tortuous/challenging vessel anatomies. To maintain proper valve leaflet coaptation, do not overinflate the deployment balloon. Appropriate antibiotic prophylaxis is recommended post-procedure in patients at risk for prosthetic valve infection and endocarditis. Additional precautions for transseptal replacement of a failed mitral valve bioprosthesis include, the presence of devices or thrombus or other abnormalities in the caval vein precluding safe transvenous femoral access for transseptal approach; and the presence of an Atrial Septal Occluder Device or calcium preventing safe transseptal access. Special care must be exercised in mitral valve replacement to avoid entrapment of the subvalvular apparatus. Safety and effectiveness have not been established for patients with the following characteristics/comorbidities: non-calcified aortic annulus; severe ventricular dysfunction with ejection fraction  $< 20\%$ ; congenital unicuspid aortic valve; pre-existing prosthetic ring in the tricuspid position; severe mitral annular calcification (MAC); severe ( $> 3+$ ) mitral insufficiency, or Gorlin syndrome; blood dyscrasias defined as leukopenia (WBC  $< 3000$  cells/mL), acute anemia (Hb  $< 9$  g/dL), thrombocytopenia (platelet count  $< 50,000$  cells/mL), or history of bleeding diathesis or coagulopathy; hypertrophic cardiomyopathy with or without obstruction (HOCM); echocardiographic evidence of intracardiac mass, thrombus, or vegetation; a known hypersensitivity or contraindication to aspirin, heparin, ticlopidine (Ticlid), or clopidogrel (Plavix), or sensitivity to contrast media, which cannot be adequately premedicated; significant aortic disease, including abdominal aortic or thoracic aneurysm defined as maximal luminal diameter 5 cm or greater, marked tortuosity (hyperacute bend), aortic arch atheroma (especially if thick [ $> 5$  mm], protruding, or ulcerated) or narrowing (especially with calcification and surface irregularities) of the abdominal or thoracic aorta, severe "unfolding" and tortuosity of the thoracic aorta; access characteristics that would preclude safe placement of the Edwards sheath, such as severe obstructive calcification or severe tortuosity; bulky calcified aortic valve leaflets in close proximity to coronary ostia; a concomitant paravalvular leak where the failing prosthesis is not securely fixed in the native annulus or is not structurally intact (e.g., wireform frame fracture, annuloplasty ring dehiscence); or a partially detached leaflet of the failing bioprosthesis that in the aortic position may obstruct a coronary ostium. For Left axillary approach, a left subclavian takeoff angle  $\sim 90^\circ$  from the aortic arch causes sharp angles, which may be responsible for potential sheath kinking, subclavian/axillary dissection and aortic arch damage. For left/right axillary approach, ensure there is flow in Left Internal Mammary Artery (LIMA)/Right Internal Mammary Artery (RIMA) during procedure and monitor pressure in homolateral radial artery. Residual mean gradient may be higher in a "THV-in-failing prosthesis" configuration than that observed following implantation of the valve inside a native aortic annulus using the same size device. Patients with elevated mean gradient post procedure should be carefully followed. It is important that the manufacturer, model and size of the preexisting prosthesis be determined, so that the appropriate valve can be implanted and a prosthesis-patient mismatch be avoided. Additionally, pre-procedure imaging modalities must be employed to make as accurate a determination of the inner diameter as possible.

**Potential Adverse Events:** Potential risks associated with the overall procedure, including potential access complications associated with standard cardiac catheterization, balloon valvuloplasty, the potential risks of conscious sedation and/or general anesthesia, and the use of angiography: death; stroke/transient ischemic attack, clusters, or neurological deficit; paralysis; permanent disability; respiratory insufficiency or respiratory failure; hemorrhage requiring transfusion or intervention; cardiovascular injury including perforation or dissection of vessels, ventricle, atrium, septum, myocardium, or valvular structures that may require intervention; pericardial effusion or cardiac tamponade; thoracic bleeding; embolization including air, calcific valve material, or thrombus; infection including septicemia and endocarditis; heart failure; myocardial infarction; renal insufficiency or renal failure; conduction system defect which may require a permanent pacemaker; arrhythmia; retroperitoneal bleed; arteriovenous (AV) fistula or pseudoaneurysm; reoperation; ischemia or nerve injury or brachial plexus injury; restenosis; pulmonary edema; pleural effusion; bleeding; anemia; abnormal lab values (including electrolyte imbalance); hypertension or hypotension; allergic reaction to anesthesia, contrast media, or device materials; hematoma; syncope; pain or changes (e.g., wound infection, hematoma, and other wound care complications) at the access site; exercise intolerance or weakness; inflammation; angina; heart murmur; and fever. Additional potential risks associated with the use of the valve, delivery system, and/or accessories include: cardiac arrest; cardiogenic shock; emergency cardiac surgery; cardiac failure or low cardiac output; coronary flow obstruction/transvalvular flow disturbance; device thrombosis requiring intervention; valve thrombosis; device embolization; device migration or malposition requiring intervention; left ventricular outflow tract obstruction; valve deployment in unintended location; valve stenosis; structural valve deterioration (wear, fracture, calcification, leaflet tear/tearing from the stent posts, leaflet retraction, suture line disruption of components of a prosthetic valve, thickening, stenosis); device degeneration; paravalvular or transvalvular leak; valve regurgitation; hemolysis; device explants; nonstructural dysfunction; mechanical failure of delivery system and/or accessories; and non-emergent reoperation.

#### **Edwards Crimper**

**Indications:** The Edwards crimper is indicated for use in preparing the Edwards SAPIEN 3 transcatheter heart valve, Edwards SAPIEN 3 Ultra transcatheter heart valve, and the Edwards SAPIEN 3 Ultra RESILIA transcatheter heart valve for implantation.

**Contraindications:** There are no known contraindications.

**Warnings:** The device is designed, intended, and distributed for single use only. **Do not resterilize or reuse the device.** There are no data to support the sterility, nonpyrogenicity, and functionality of the device after reprocessing. Do not mishandle the device. Do not use the device if the packaging or any components are not sterile, have been opened or are damaged, or the expiration date has elapsed.

**Precautions:** For special considerations associated with the use of the Edwards crimper prior to THV implantation, refer to the THV Instructions for Use.

**Potential Adverse Events:** There are no known potential adverse events associated with the Edwards crimper.

**CAUTION: Federal (United States) law restricts these devices to sale by or on the order of a physician.**

#### **References**

1. Mack MJ, Leon MB, Thourani VH, et al. Transcatheter aortic-valve replacement with a balloon-expandable valve in low-risk patients. *N Engl J Med.* 2019;380(18):1695-1705.
2. Mack MJ, Leon MB, Thourani P, et al; PARTNER 3 investigators. Transcatheter aortic-valve replacement in low-risk patients at five years. *N Engl J Med.* 2023;389(21):1949-1960.
3. Williams MR, Jilalhawani H, Makkar R, et al. The PARTNER 3 Bicuspid Registry for transcatheter aortic valve replacement in low-surgical-risk patients. *JACC Cardiovasc Interv.* 2022;15(5):523-532.
4. Eng MH, Abbas AE, Hahn RT, et al. Real world outcomes using 20 mm balloon expandable SAPIEN 3/ultra valves compared to larger valves (23, 26, and 29 mm) – a propensity matched analysis. *Catheter Cardiovasc Interv.* 2021;98(6):1185-1192.
5. Nazif TM, Cahill TJ, Daniels D, et al. Real-world experience with the SAPIEN 3 Ultra transcatheter heart valve: a propensity-matched analysis from the United States. *Circ Cardiovasc Interv.* 2021;14(9):e010543.
6. Pibarot P, Ternacle J, Jaber WA, et al. Structural deterioration of transcatheter versus surgical aortic valve bioprostheses in the PARTNER-2 trial. *J Am Coll Cardiol.* 2020;76(16):1830-1843.
7. Okuno T, Tomii D, Lanz J, et al. 5-year outcomes with self-expanding vs balloon-expandable transcatheter aortic valve replacement in patients with small annuli. *JACC Cardiovasc Interv.* 2023;16(4):429-440.
8. Tarantini G, Nai Fovino L, Le Prince P, et al. Coronary access and percutaneous coronary intervention up to 3 years after transcatheter aortic valve implantation with a balloon-expandable valve. *Circ Cardiovasc Interv.* 2020;13(7):e008972.
9. Kaneko T, Makkar RR, Krishnaswami A, et al. Valve-in-surgical-valve with SAPIEN 3 for transcatheter aortic valve replacement based on Society of Thoracic Surgeons predicted risk of mortality [published correction appears in *Circ Cardiovasc Interv.* 2021;14(7):e000083]. *Circ Cardiovasc Interv.* 2021;14(5):e010288.
10. Makkar RR, Kapadia S, Chakravarty T, et al. Outcomes of repeat transcatheter aortic valve replacement with balloon-expandable valves: a registry study. *Lancet.* 2023;402(10412):1529-1540.
11. Ochiai T, Oakley L, Sekhon N, et al. Risk of coronary obstruction due to sinus sequestration in redo transcatheter aortic valve replacement. *JACC Cardiovasc Interv.* 2020;13(22):2617-2627.

Edwards, Edwards Lifesciences, the stylized E logo, Edwards SAPIEN, Edwards SAPIEN 3, Edwards SAPIEN 3 Ultra, INSPIRIS, INSPIRIS RESILIA, PARTNER, PARTNER II, PARTNER 3, RESILIA, SAPIEN, SAPIEN 3, and SAPIEN 3 Ultra are trademarks of Edwards Lifesciences Corporation or its affiliates. All other trademarks are the property of their respective owners.

© 2024 Edwards Lifesciences Corporation. All rights reserved. PP--US-8992 v1.0

Edwards Lifesciences • One Edwards Way, Irvine, CA 92614 USA • edwards.com



Edwards